



Assembly Instructions for ACES Humaniac™ Animal Stretcher

Parts:

1 x Stretcher Frame	2 x Small Washers
1 x Stretcher Material	4 x Large Washers
2 x Phillips Head Screws	2 x Wing Nuts
2 x Wheels	2 x Split Pins
1 x Wheel Bar	2 x Eyebolt Assembly (Eyebolt, Wing nut, Washer)
1 x Shoulder Strap	

Steps:

- Kit will come with stretcher material already attached to frame.
- Attach frame tubing to create full stretcher.
- If desired insert Phillips Head Screws into holes in middle of frame to anchor frame assembly (Fig 1).
- Pull stretcher material taut, using the velcro strips on both ends to anchor
- Insert wheel bar into machined holes on the end of stretcher (the wheel bar pegs will only fit on one side of stretcher) and attach with wing nuts and small washers (Fig. 2a & 2b)..
- Attach wheels (large washer – wheel – large washer) to the sides of the wheel bar, use the split pins to keep the wheels in place (Fig. 3).
- Attach eyebolt assembly on opposite side of stretcher; make sure that the eye of the bolt is on the same side as the restraint flap of the stretcher (Fig. 4a & 4b).
- Strap will clip to the eyebolts allowing a single person to pull the loaded stretcher.
- Adjust strap to desired length.



Fig. 1



Fig. 2a

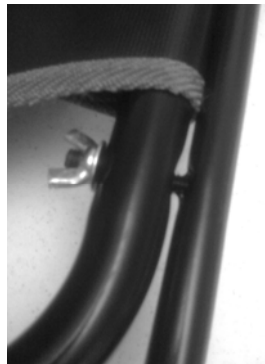


Fig. 2b



Fig 3

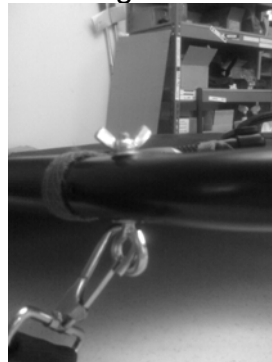


Fig 4a



Fig 4b